
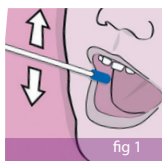



## Sampling Instructions

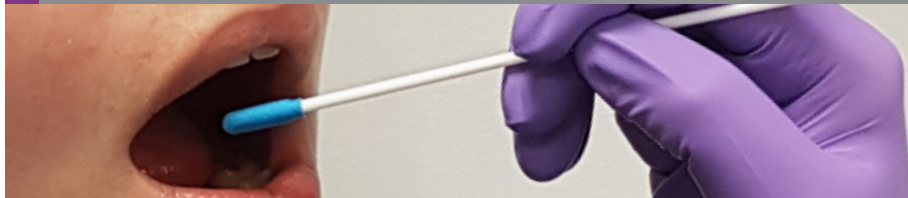

### 1. SWAB FOR TEN SECONDS


Rub all sides of the swab:

- Inside both cheeks
- Between the lips and teeth

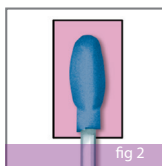


### 2. PRESS SWAB FIRMLY ONTO THE PINK PAPER FOR FIVE SECONDS

- Do not rub

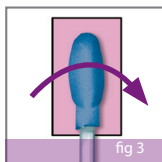

### 3. TURN SWAB OVER AND PRESS FOR A FURTHER FIVE SECONDS

- In the same place as before, do not rub

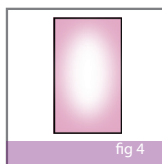

An area of the pink paper, the size of the swab, should now have turned white. This confirms the sample has been taken correctly.


Sampler signs here

ALL SECTIONS <b>MUST</b> BE COMPLETED	ALL SECTIONS <b>MUST</b> BE COMPLETED
SAMPLER'S NAME: _____	DONOR'S NAME: _____
SAMPLE DATE: <input type="text"/> <input type="text"/> <input type="text"/> <input type="text"/> <input type="text"/> <input type="text"/>	BIRTHDATE: <input type="text"/> <input type="text"/> <input type="text"/> <input type="text"/> <input type="text"/> <input type="text"/>
SAMPLER'S SIGNATURE: _____	

It is essential that the Sampler **SIGNS** the collection card. Please use a **BALL POINT PEN**. After sampling close the cover on the collection card and place it back into its small plastic bag. Discard the swab.

**Do not touch the swab tip or the pink paper with your fingers and do not replace the swab back into the Donor's mouth after contact with the pink paper.**

## Identification requirements

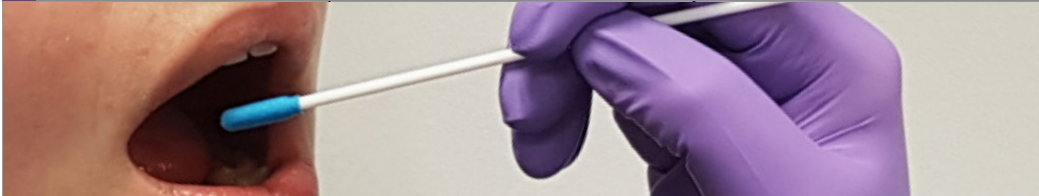

We cannot process a sample without:

- Two photographs of the Donor
- A completed and signed Sample Declaration Form
- A completed and signed Collection Card

### Photographs

Photographs must be a good likeness, full face and without a hat or sunglasses.

The back of the photograph must be signed, by the sampler, and stapled to the Sample Declaration Form.

### Sample Declaration Form

All relevant sections must be completed in BLOCK CAPITALS. We require a signature from the donor, the person giving consent (if the donor is under 18) and the sampler.

### Returning the samples

The sampler who confirms the identification and signs the Sample Declaration Form

**MUST:**

- Take and package the sample
- Seal all samples and Sample Declaration Form(s) in the tamper evident mailing envelope for return to Cellmark.

**Thank you for your assistance.**

If you are uncertain about any part of the procedure please telephone Cellmark on:

**Customer Services +44 (0)1235 528000**